

Bronocide™ **ULTRA-10** ULTRA SONIC CLEANER

**NON
CORROSIVE**

**Specially formulated Alkaline disinfecting cleanser
for ultrasonic and universal application**

CONCENTRATED ALKALINE CLEANSER

RESIDUE FREE • BIODEGRADABLE • 100% SOLUBLE



ULTRASONIC & UNIVERSAL APPLICATION

USE:

Specially formulated for root canal instruments/burs/diamond grinders/scalpels/mirrors/tweezers/elevators etc. also crowns/bridges/inlays/dentures and all items made of glass and plastic.

NOTE:

Virtually eliminates scrubbing by hand and totally removes dried blood and saliva from instruments in under 6 minutes.

Available in: 1 Litre

INSTRUCTIONS FOR USE:

1. Dilute 1 part Ultra-10 with 19 parts water for use. (5% Solution)
2. Treat instruments etc. in the ultrasonic bath for 3 - 6 minutes depending on degree of contamination.
3. Rinse instruments in clean water after treatment, and sterilise as required.

CONTAINS

Benzalkonium chloride, rust inhibitor, chelating agent

CAUTION

Keep out of reach of children. In case of accidental eye contact, rinse with plenty of water.

STORE BELOW 25°C

FIRST AID MEASURES

Eyes: Wash immediately with copious amounts of water and obtain medical attention.

Skin: Prolonged contact may cause irritation with sensitive skin types. Avoid prolonged skin contact and wash off with water if irritation occurs.

Ingestion: Rinse mouth and drink 2 glasses of water or milk. If a large amount is ingested and symptoms develop, seek medical attention. Do not induce vomiting.

PERSONAL PROTECTIVE EQUIPMENT:

Eyes: When the product is used as directed, personal protective equipment is not required. However, the product is an eye irritant, and as such it should be handled in such a way as to avoid eye contact.

Hand/Skin: Prolonged skin contact should be avoided, and if an application dictates otherwise, the use of protective gloves or clothing is recommended.

Respiration: No special protection is required.