

**NON
CORROSIVE**

Bronocide™ VACU-CLEANSER

Concentrated formulation for effective disinfection & cleansing

SANITIZING VACUUM SYSTEM CLEANSER

NON-FOAMING • FREE RINSING



USE:

In all vacuum system applications. For effective disinfection and cleaning use at least once a day.

NOTE:

Product is non-foaming / non-corrosive, free rinsing, non-corrosive, residue-free 100% soluble and biodegradable

Available in: 1 kg Jar

INSTRUCTIONS FOR USE:

1. Add 4 tablespoons of VACU-CLEANSER to 5 litres of warm water.
2. Stir until completely dissolved.
3. Draw solution through the high volume hose and saliva ejector.

CONTAINS

Surfactants, alkalisng agent, flocculant, sodium chloride.

CAUTION

Keep out of reach of children. In case of accidental eye contact, rinse with plenty of water.

STORE BELOW 25°C

FIRST AID MEASURES

Eyes: Wash immediately with copious amounts of water and obtain medical attention.

Skin: Prolonged contact may cause irritation with sensitive skin types. Avoid prolonged skin contact and wash off with water if irritation occurs.

Ingestion: Rinse mouth and drink 2 glasses of water or milk. If a large amount is ingested and symptoms develop, seek medical attention. Do not induce vomiting.

PERSONAL PROTECTIVE EQUIPMENT:

Eyes: When the product is used as directed, personal protective equipment is not required. However, the product is an eye irritant, and as such it should be handled in such a way as to avoid eye contact.

Hand/Skin: Prolonged skin contact should be avoided, and if an application dictates otherwise, the use of protective gloves or clothing is recommended.

Respiration: No special protection is required.